POLSKI

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## IRIDEX MoistAir™ Humidifying Chamber - REF 200-10

**English**
**Instructions for use - Air Line**

**Indications:** The MoistAir device is used for infusion of fluid and air during vitrectomy.

**Intended use:** MoistAir connects to any standard tubing set and provides humidified air to the eye during fluid air exchange. Studies have shown that humidified air may substantially reduce the dehydrating effects of dry air, delay lens feathering, protect corneal endothelium, and may prevent visual field defects after macula hole surgery. For published clinical studies contact IRIDEX.

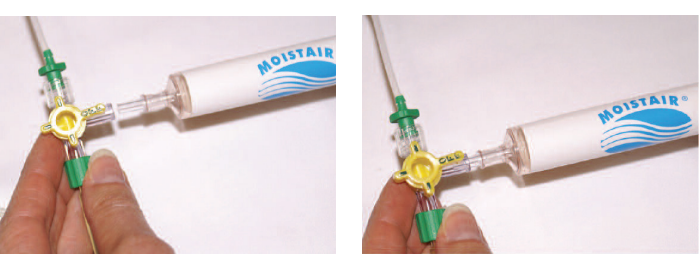
**IMPORTANT:** Prior to air exchange, verify that the air pump pressure is sufficient to ensure adequate IOP.

**Directions:**

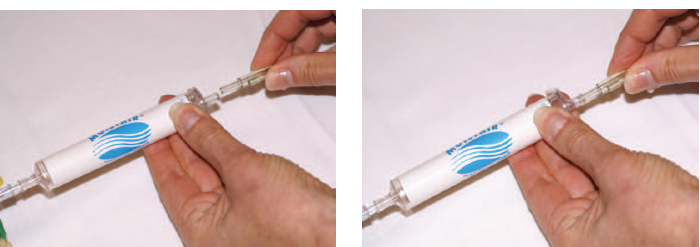
- Set up the air line set per manufacturer’s directions
- Connect the air line to the air machine
- Connect MoistAir as shown below:

**Step 1:** Fully saturate MoistAir chamber by flushing sterile fluid.


**Step 2:** With stopcock closed, insert MoistAir into the 3 way stopcock as shown. (Figure A, B)

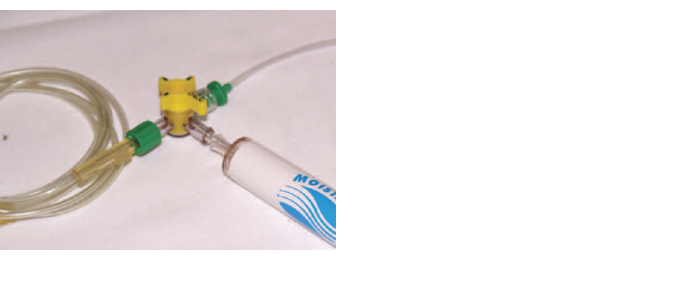**Step 3:** Insert the air line into the rear of the MoistAir as shown. (Figure C, D)

**Step 4:** Connect infusion cannula into the stopcock. (Figure E)

  Em ergo Europe Molenstraat 15, 2513 BH The Hague, The Netherlands Tel: (31) (0) 70 345-8570, Fx: (31) (0) 70 346-7299

 **IRIDEX**

 IRIDEX Corporation 1212 Terra Bella Ave, Mountain View, CA, USA 94043 Tel: 650-962-8100 Fax: 650-962-0486 E: info@iridex.com

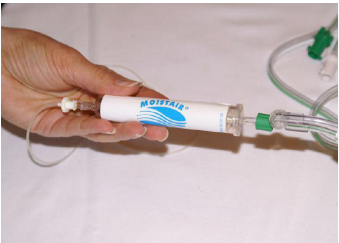
Caution: Federal law restricts this device to sale by or on the order of a physician Made in USA US Patents: US: 5,997,498 and 6,726,666; CA - 2,331,837; AU - 759,193; and JP - 4,149,670

     0086


P/N 21070 Rev C 5/2012

**Instructions for use - Infusion Line**

MoistAir Humidifying Chamber with infusion line connection. (Figure F)

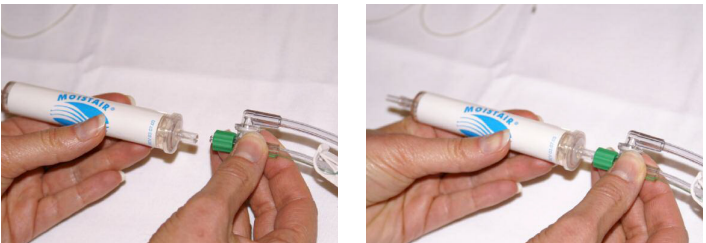**Step 1:** Connect MoistAir to infusion line. (Figure G, H)

**Step 2:** Connect infusion cannula to MoistAir. (Figure I, J)

**Step 3:** Prime line according to manufacturer’s instructions.

**WARNING:**The MoistAir Humidifying Chamber is a sterile single-use device. It is intended for use on one patient during a single procedure. It is not intended to be reprocessed (cleaned, disinfected/sterilized) and used on another patient. No reuse processes have been validated by the manufacturer. The reuse of a single-use medical device can compromise the safety and health of patients, users or third parties. Reduce risks the introduction of contaminants and/or microbes into the body of a patient as a result of bio-hazardous contamination from its first use.

Inspect the packaging prior to use: DO NOT USE IF THE PACKAGE IS DAMAGED OR IF THE STERILE BARRIER IS COMPROMISED. VERIFY THE SUITABILITY OF ALL CONNECTIONS; DO NOT USE IF ANY CONNECTION IS NOT SUITABLE.

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